



Saving InVivoDental images to a .dcm or .inv file you can send to IDT

Step 1

- Double click the icon on your desktop to launch the InVivo software.

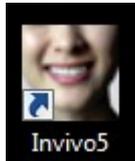


- Select the patient from the list or **Browse File** to locate the file you want.

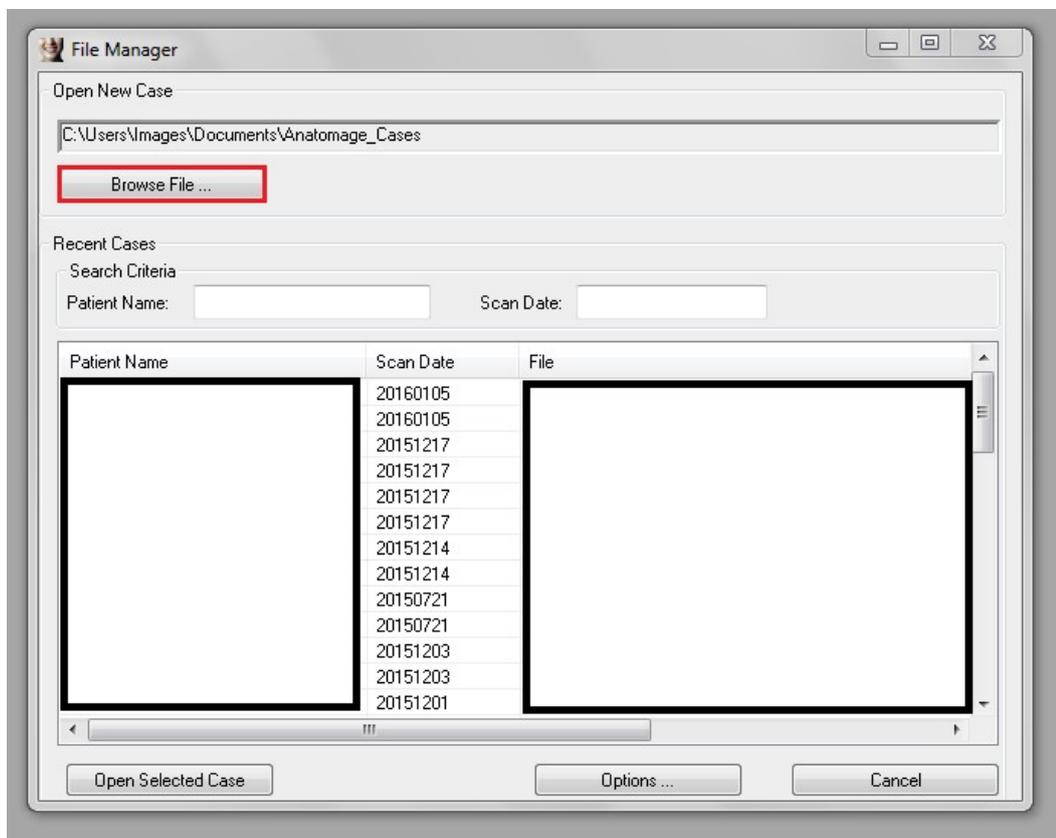


Step 2

- To export .dcm or .inv files go to the **File** menu and select the **Save As...** option
- **Do not use** 'Export DICOM & Viewer' as this option creates a .exe file that cannot be used for further processing.

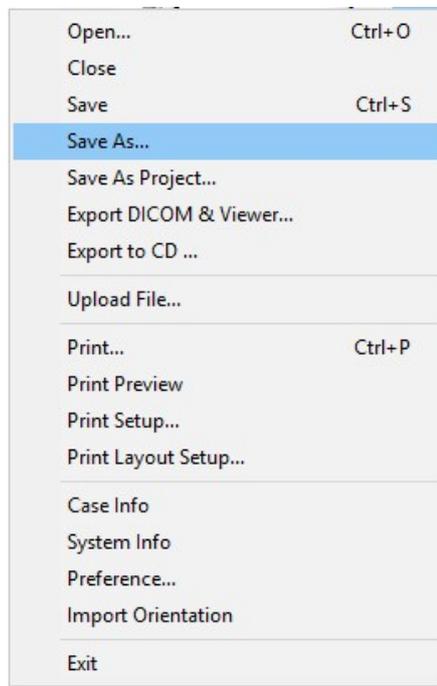


Step 3

- Select **Invivo** or **Single File DICOM** and click **OK**.

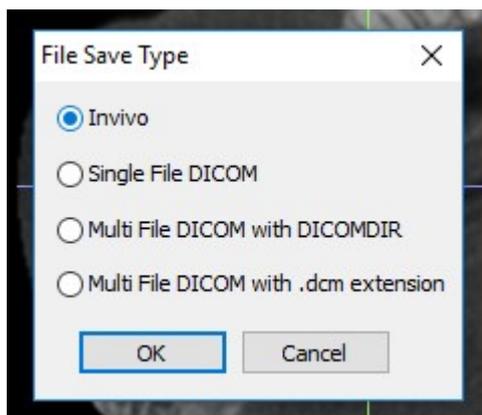


Step 4

- A window will automatically open
- Navigate to your Desktop and Save the file using the Patient's Name:

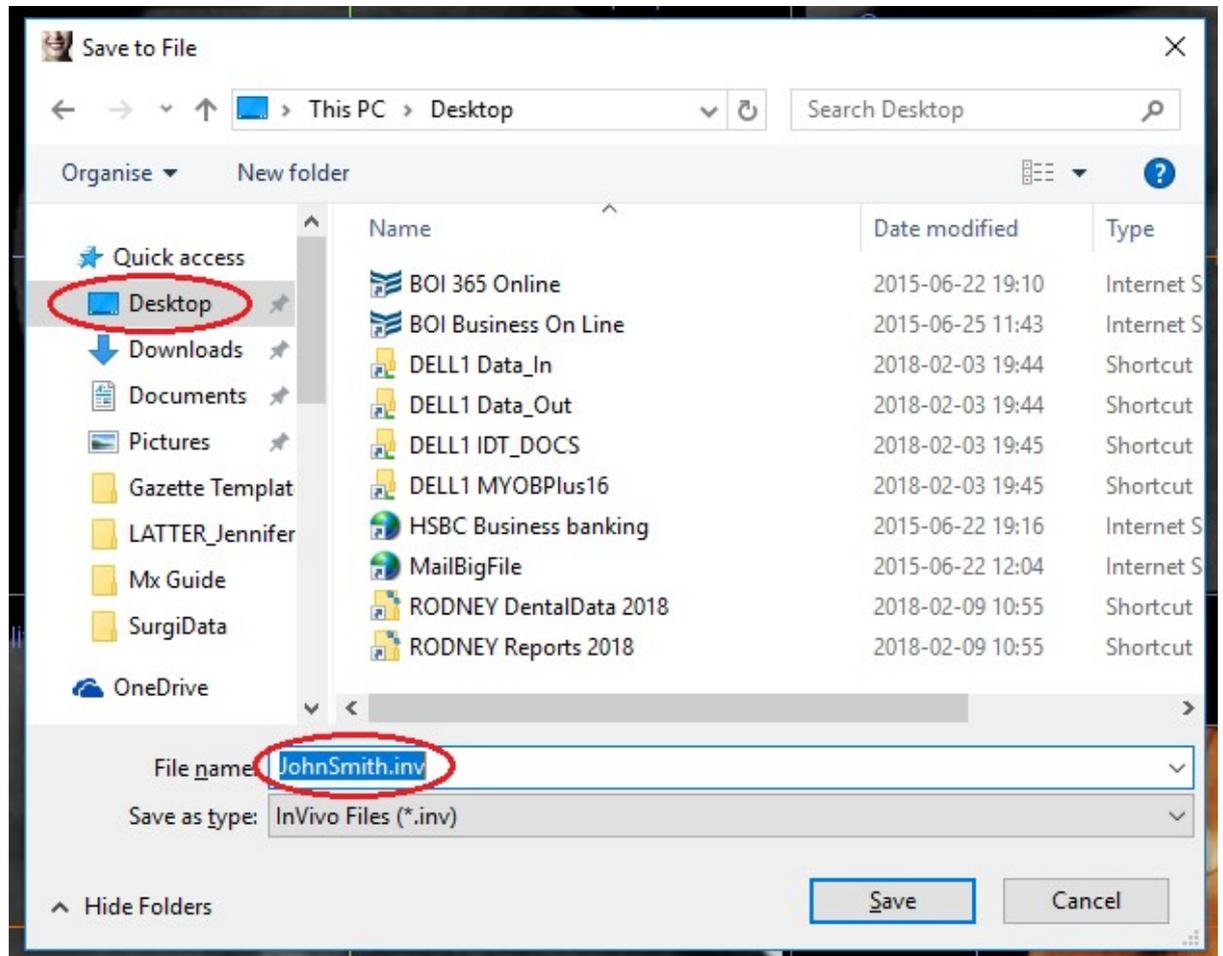


Step 5

- If prompted, Select **Full** and click **Save**

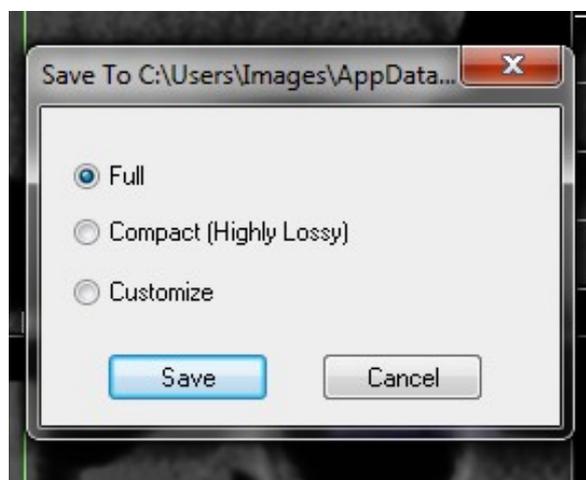

Step 6

- The following message will appear once all of the images have been successfully exported. Just click **OK** to proceed.

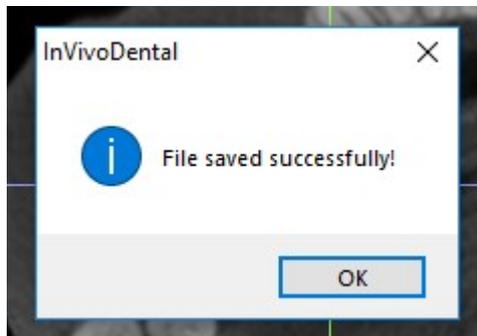

The .inv file or .dcm that you saved on your Desktop can be sent safely and securely to IDT Scans in a number of ways:


- You can click on a link on our website www.idtscans.com, OR
- If you have an sFTP account you can drag the file to the "To_IDT" folder, OR
- You can attach the file while you are booking the case in for processing.

To send the .dcm or .inv file to IDT via the Home Page of our website:

Step 7


Go to www.idtscans.com

If you haven't registered, click on "**Upload Your File Now**":

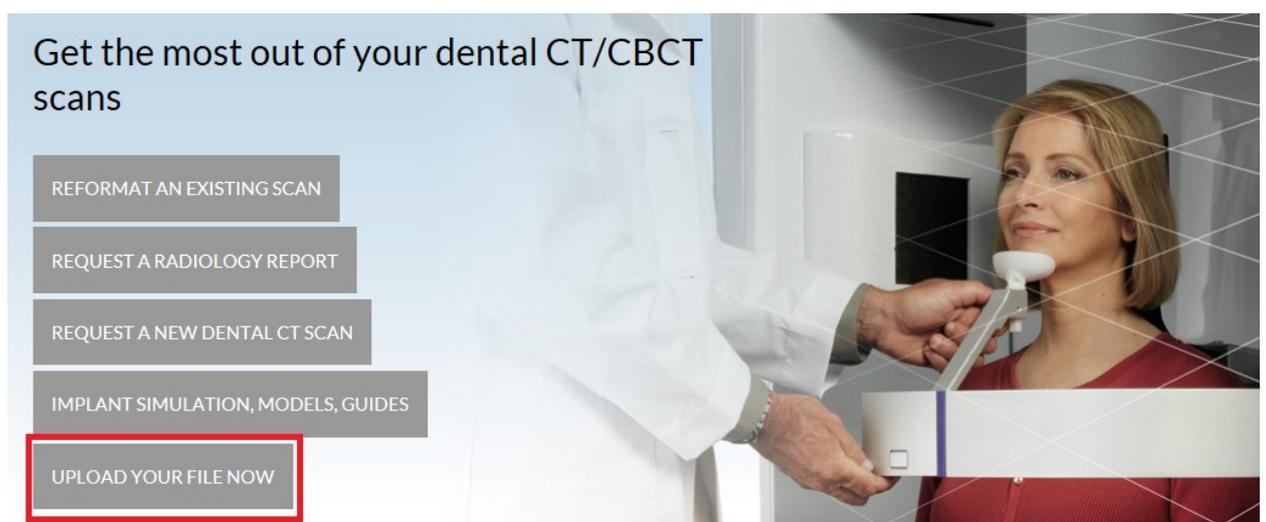

Step 8

On the next page, click on “**Upload a Single File**”:

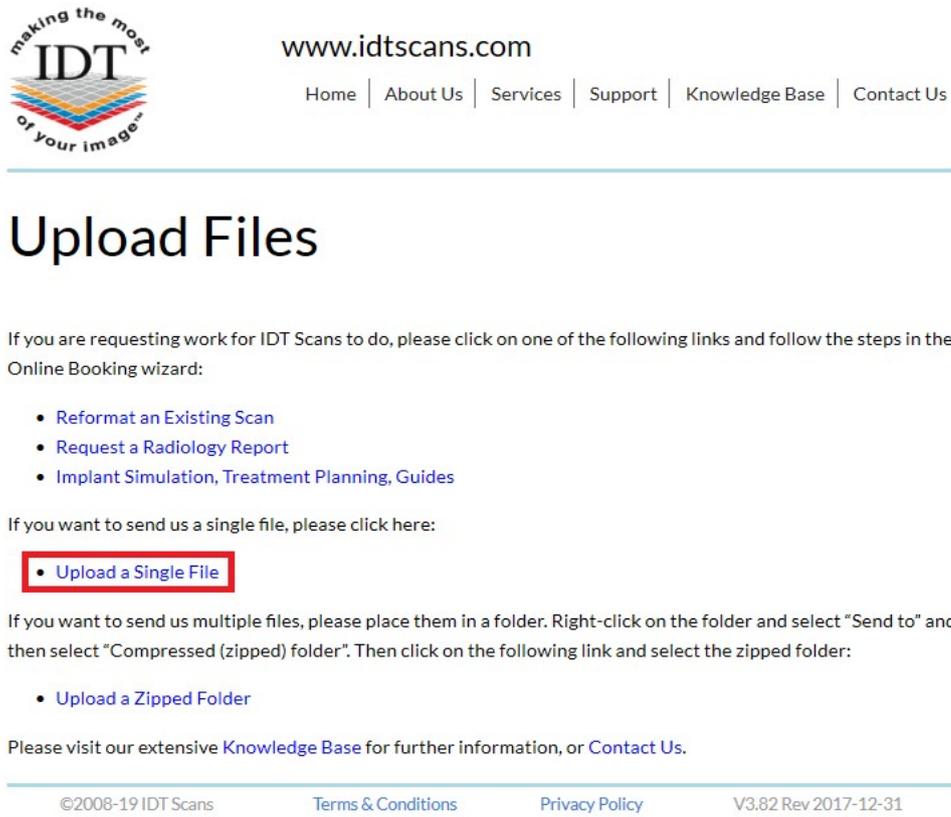

making the most
IDT
of your image™

www.idtscans.com

Home | About Us | Services | Support | Knowledge Base | Contact Us

Upload Files

If you are requesting work for IDT Scans to do, please click on one of the following links and follow the steps in the Online Booking wizard:

- [Reformat an Existing Scan](#)
- [Request a Radiology Report](#)
- [Implant Simulation, Treatment Planning, Guides](#)

If you want to send us a single file, please click here:

- [Upload a Single File](#)

If you want to send us multiple files, please place them in a folder. Right-click on the folder and select “Send to” and then select “Compressed (zipped) folder”. Then click on the following link and select the zipped folder:

- [Upload a Zipped Folder](#)

Please visit our extensive [Knowledge Base](#) for further information, or [Contact Us](#).

©2008-19 IDT Scans [Terms & Conditions](#) [Privacy Policy](#) V3.82 Rev 2017-12-31

Step 9

Follow the 3 steps below to attach and send your file. If you cannot see the **Browse** button please install Adobe Flash Player from <http://get.adobe.com/flashplayer>



making the most
IDT
of your image™

www.idtscans.com

Mail a Big File to idtscans.com

In **Secure Mode** 🔒

- 1 Attach Files** (maximum file size 2GB)
[Redacted]
- 2 Your Email Address**
[Redacted]@idtscans.com
- 3 Message**
it is easy to send data to
IDT Scans :)

Developed & Hosted by Dessol | ©2005 - 2017 | Powered By MailBigFile.com | [Switch to Multi Uploader](#)   
MailBigFile ® registered trademark of MailBigFile Ltd