


Installing Simplant View or OneShot from a CD/DVD or USB stick you received from IDT

To Install Simplant View or Simplant OneShot


Step 1

Insert CD/DVD into drive.

The IDT Simplant Install Launcher will start automatically:

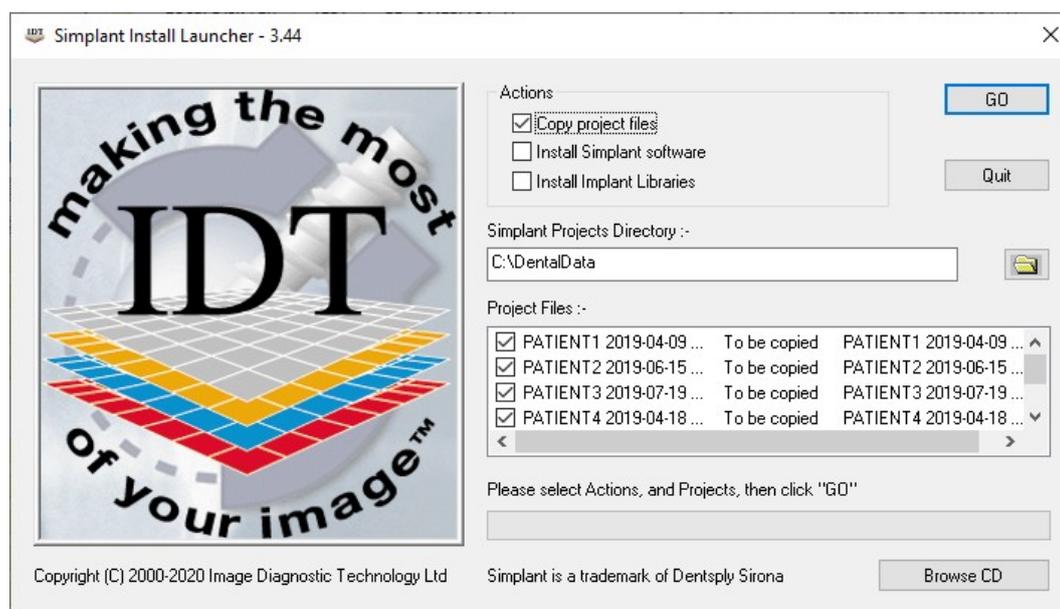


If the Install Launcher does not start automatically, use File Explorer to navigate to the root of the CD/DVD or memory stick and double-click "Setup.exe":

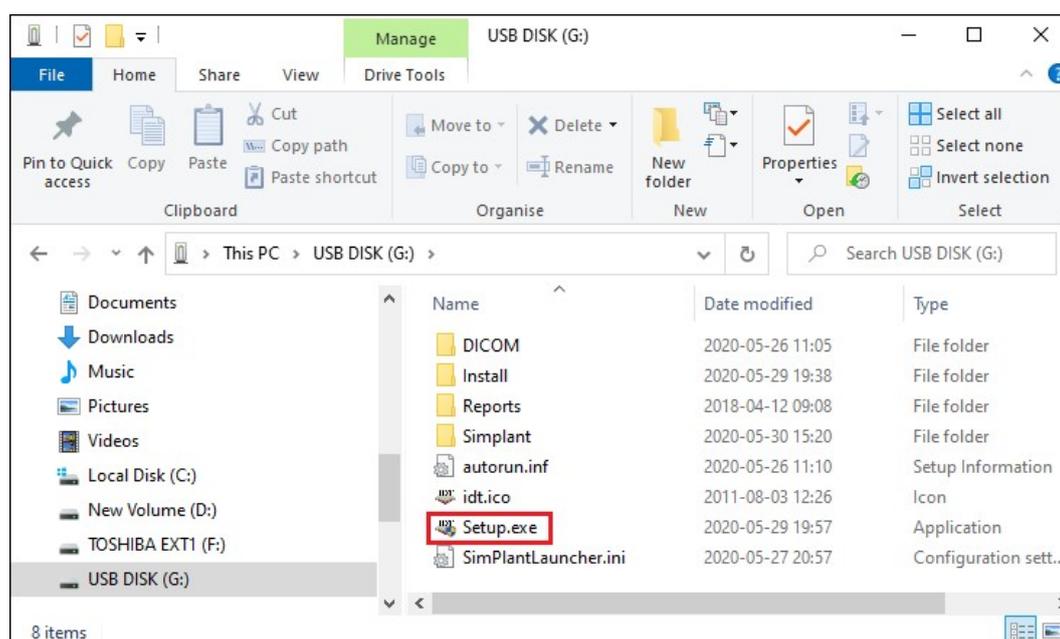

Step 2


Tick the box next to “Copy project files” if you want to copy Simplant project files to the **C:\DentalData** folder on your computer. We recommend storing all Simplant Projects in the **C:\DentalData** folder. However, you may browse to a different folder using the button shown.

You can untick the boxes next to any projects that you don't want to copy.

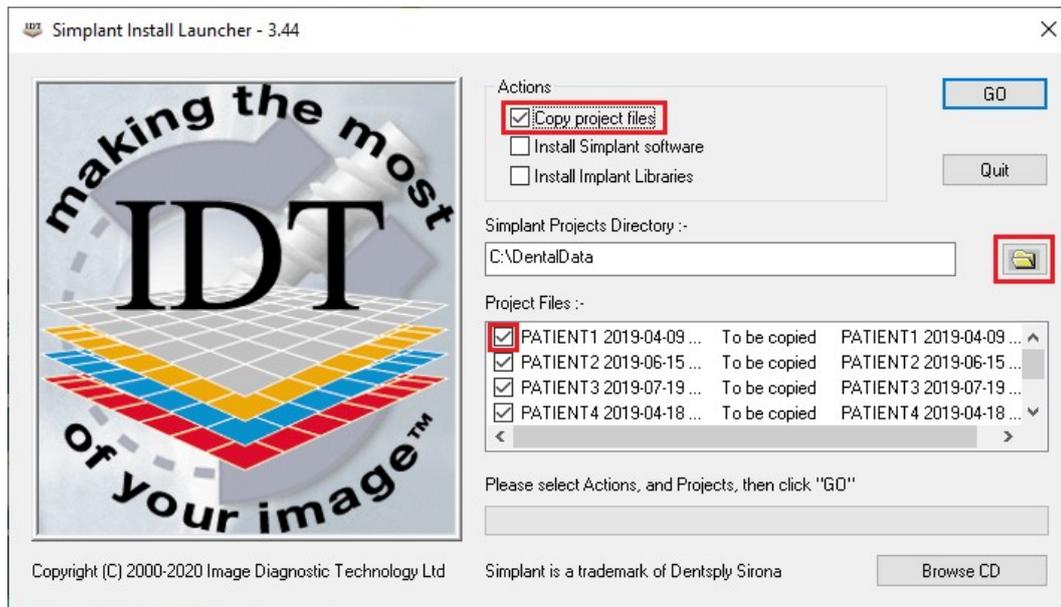

Step 3

Tick the box next to “Install SimPlant software” if you want to install Simplant View or OneShot on your computer. Leave it unticked if you already have Simplant on your computer. For Simplant OneShot you should install the Implant Libraries as well.

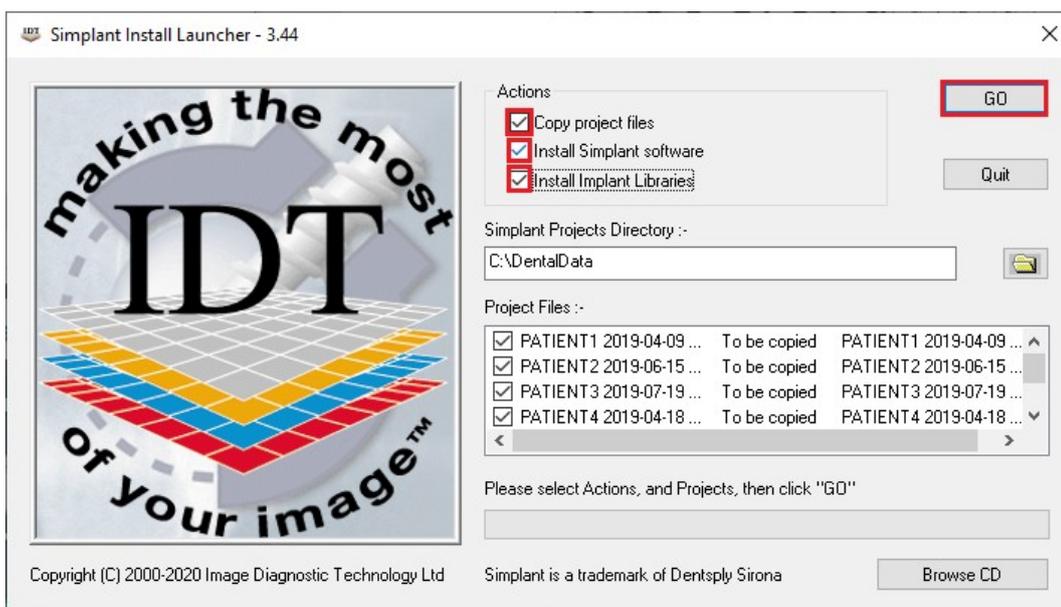

Please click the **GO** button when you are ready to proceed.

Step 4


The Simplant project files will be copied to the folder selected (usually **C:\DentalData**). If any project file already exists, you can choose to overwrite it or rename it.

Step 5

The Simplant software may take some time to start installing. Please wait till it starts installing, then follow the onscreen instructions. Similarly for the Implant Libraries (if selected).

Step 6

When installation is complete you will see this message:

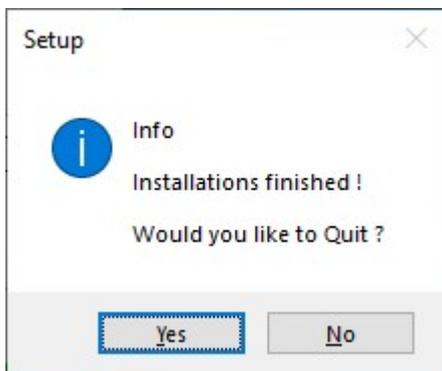

Click **Yes** to terminate IDT Simplant Install Launcher.


Step 7

If installation was successful you will see a Simplant icon on your Desktop. Launch Simplant by double-clicking the Simplant icon.



Step 8

From the **File** menu select **Open Files ...**

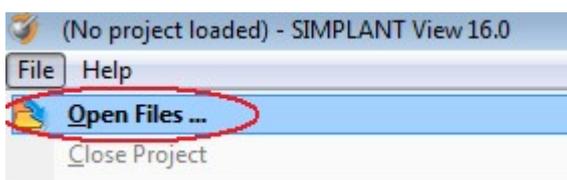


Step 9

A dialog box will open and you will see your patient listed. Double-click on your patient to view the images.

Step 10

If there are **DICOM files** or **Radiology Reports** on the CD/DVD/USB stick you can access them by starting Simplant Install Launcher and clicking "**Browse CD**":

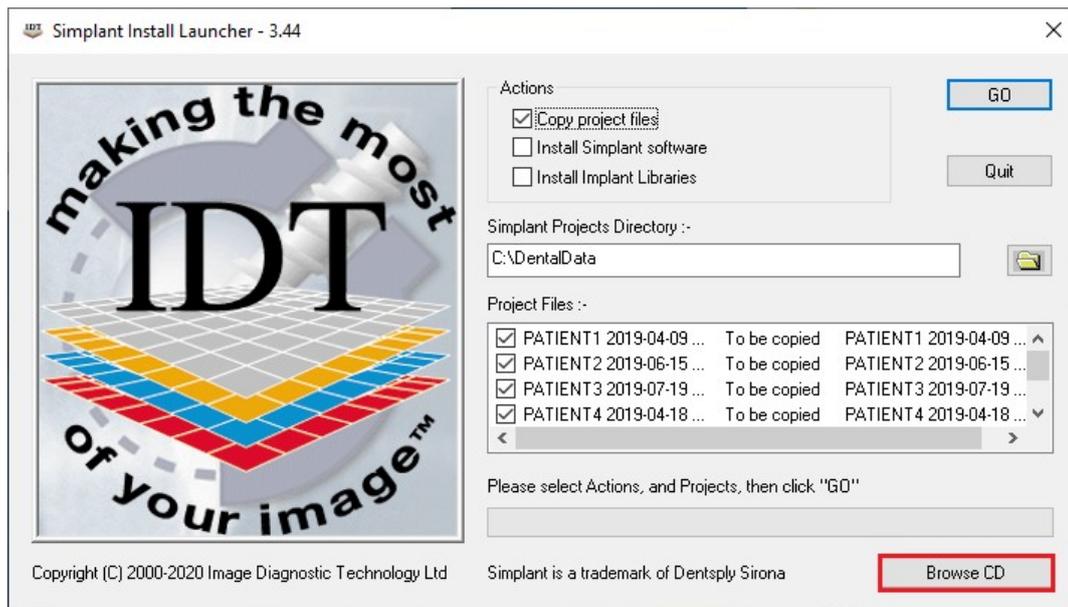

Created 2015-02-01 RAR

Revised 2020-03-08 DF/RAR

Last Revised 2020-06-01 RAR