

## Downloading and installing Simplant OneShot

Simplant OneShot is free software that can be used to view CT Scans, take basic measurements, plan implants and order surgical drill guides.

**The CT images must be in Simplant OneShot format – otherwise Simplant OneShot will not display them.**

If you already have Simplant OneShot installed on your computer you may skip to Step 7.

### Step 1


Go to [www.idtscans.com](http://www.idtscans.com) and click **Support** at the top of the Home Page.



The screenshot shows the IDT website homepage. At the top left is the IDT logo with the tagline "making the most of your image". To the right of the logo is the website URL "www.idtscans.com" and a navigation menu with links for "Home", "About Us", "Services", "Support" (highlighted with a red box), "Knowledge Base", and "Contact Us". Below the navigation is a large banner image of a dental professional in a white coat using a handheld device on a patient's face. On the left side of the banner, there is a list of five buttons: "REFORMAT AN EXISTING SCAN", "REQUEST A RADIOLOGY REPORT", "REQUEST A NEW DENTAL CT SCAN", "IMPLANT SIMULATION, MODELS, GUIDES", and "UPLOAD YOUR FILE NOW". At the bottom of the banner, there is a dark bar with the text "Choose a scanning site in the UK or Ireland".

### Step 2

Choose 'Software'

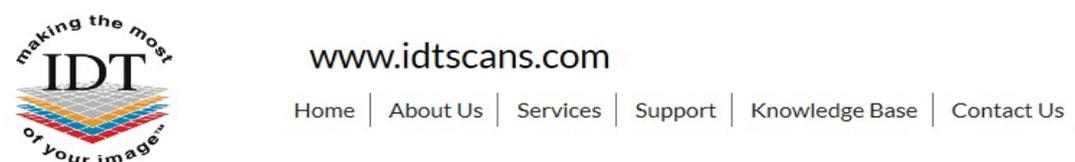

The screenshot shows the IDT website support page. At the top left is the IDT logo with the tagline "making the most of your image". To the right of the logo is the website URL "www.idtscans.com" and a navigation menu with links for "Home", "About Us", "Services", "Support", "Knowledge Base", and "Contact Us". Below the navigation is a large heading "Support".

Please click on the links to access the Support pages:

- [Frequently Asked Questions](#)
- [Software](#)
- [Downloads](#)

### Step 3

Choose 'Simplant OneShot'



www.idtscans.com

Home | About Us | Services | Support | Knowledge Base | Contact Us

## Software

For information about the Simplant software family please click [here](#).

To download Simplant software please click on the following links. You may have to scroll down several pages to find the version you need. Some files are very large and may take a long time to download.

- [Simplant Pro](#)
- [Simplant View](#)
- [Simplant OneShot](#)

### Step 4

Scroll down to find **Simplant Installers** then **Simplant 17** then **OneShot**.

Click on <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupEN.exe> to install the English version.

### Simplant 17

View:

- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTViewSetupFR.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTViewSetupEN.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTViewSetupIT.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTViewSetupES.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTViewSetupDE.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTViewSetupRU.exe>

OneShot:

- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupFR.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupEN.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupIT.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupES.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupDE.exe>
- <http://link.simplant.com/dentalsoftware/SIMPLANT17/SIMPLANTOneShotSetupRU.exe>

### Step 5

Follow the on-screen instructions to install Simplant OneShot software.

## Step 6

After installing Simplant OneShot you need to install the Implant Library. Scroll to the end of the Simplant Installers page and click on **Simplant Implant Library**. Follow the on-screen instructions to install the Simplant Implant Library  
**WARNING – very large file – may take a long time to download.**

Simplant Implant Library (only needed for Simplant 17 or older)

- Implant Library 5.0  
<http://link.simplant.com/dentalsoftware/PreviousLibrary/ImplantLibrariesSetup.exe>
- Latest Implant Library  
<http://link.simplant.com/dentalsoftware/ImplantLibrariesSetup.exe>

## Step 7

Simplant OneShot looks for images in the **C:\DentalData** folder.  
If you want to open a Simplant file using Simplant OneShot you need to save the Simplant file to the **C:\DentalData** folder first.  
Please see the document “**Downloading and Saving Simplant files**” for instructions how to do this.


## Step 8

Launch Simplant OneShot by double-clicking the Simplant OneShot icon on your Desktop.



## Step 9

From the **File** menu select **Open Files ...**

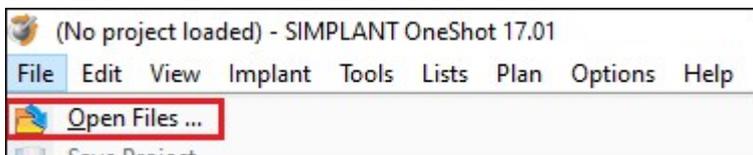

## Step 10

A dialog box will open and you will see your patient listed. Double-click on your patient to view the images.

**PLEASE NOTE: Simplant OneShot can only display files that are in Simplant OneShot format. If your file is not in Simplant OneShot format it will not be listed.**

Created by Annika Patel  
Revised 2017-12-22 DF/RAR  
Last Revised 2020-03-09 RAR